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Stand der Informationen: Juni 2018

Instructions for Use. Please read carefully

Endosgel® Chlorhexidine digluconate 0.05% EN

Sterile lubricant with physiological conductivity and supportive disinfectant properties.

Contents: Chlorhexidine digluconate 0.05%, methyl-4-hydroxybenzoate, propyl-4-hydroxybenzoate, hydroxyethyl cellulose.

Intended use: Sterile lubricant with supportive disinfectant properties and physiological conductivity to facilitate the insertion of catheters and instruments into the urethra and bladder, especially for endourological procedures in electrocoagulation and resection and for intermittent catheterisation.

Properties: **Endosgel®** adheres well to the mucosa and leaves instrument lenses with a clear view. **Endosgel®** provides supportive disinfection and offers physiological conductivity.

Side effects: Hypersensitivity can lead to skin reactions; contact dermatitis or bladder irritation can occur very rarely. In very rare cases (less than 0.01%) allergic reactions up to anaphylactic shock to chlorhexidine digluconate may occur. The physician will decide on the necessary therapeutic measures.

Contraindication: **Endosgel®** must not be used in patients with hypersensitivity/allergy to alkyl-4-hydroxybenzoates (parabens), chlorhexidine or any other constituent.

Dosage instructions: Preparation: Remove the syringe from its packaging. Before removing the blue cap, release the pressure point by pressing the plunger down. This ensures that the syringe can be easily and evenly emptied.

Application: Unless otherwise prescribed/specified,

a) For transurethral insertion of catheters and instruments: instil 6ml or 11 ml. Following the usual cleaning and disinfection of the surrounding skin and/or mucosal areas, e.g. the external genitals or the external urethral orifice, slowly instil **Endosgel®** into the urethra. The entire urethra, including the external sphincter, must be covered with a lubricating film to ensure gentle insertion of catheters and instruments. The quantity depends on the patient's anatomic conditions and/or on the procedure to be performed. For example, the disposable syringe with 6ml **Endosgel®** is suitable for use in women, while the 11ml is suitable for use in men and the 20ml for electroresection and coagulation. In addition, the use of a penis clamp is recommended in male patients.

b) for other urological procedures: instil 11ml, and where necessary another 6ml or 11ml into the body orifice. The quantity depends on the patient's anatomic conditions and/or on the procedure to be performed.

For the gel to achieve its full effectiveness, wait for 5-10 minutes.

Disposal: Please follow the relevant national regulations for disposal.

Notes and information on shelf life: Do not use **Endosgel®** after the expiry date has passed!

Discard any residual gel and do not use the empty syringe for other purposes, as this may result in transmission of germs.

Keep out of the reach of children!

Packs of 10 and 100 disposable syringes containing 6/11/20ml each.

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FARCO-PHARMA GmbH