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Farco-fill® Protect

DE

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Farco-fill® Protect

EN

Description: The medical product **Farco-fill® Protect** is a sterile solution, provided in prefilled syringes, for inflating the balloon of urinary catheters, once they are inserted into the bladder.

Farco-fill® Protect is composed of a polyethylene glycol that binds defined quantities of water, thereby reliably inflating urinary bladder catheters over extended periods.

Composition: **Farco-fill® Protect** contains: Purified water, macrogol, macrogolglycerolhydroxystearate and triclosan.

Proper use: As a result of its practical application form, **Farco-fill® Protect** can be instilled directly into the balloon port of the catheter. **Farco-fill® Protect** is designed to enable quick and easy balloon inflation. The volume instilled is dependent on the size of the balloon and the type of catheter and is normally between 5-10ml.

Information on application: Regular replacement of the indwelling catheter is imperative for hygienic reasons and should be undertaken at the intervals recommended.

Warning notice: Do not use **Farco-fill® Protect** for patients with known sensitivity towards macrogol, macrogolglycerolhydroxystearate and / or triclosan.

The disposable syringes should not be repeatedly used to avoid the risk of transmission of pathogens.

Farco-fill® Protect should be only used to inflate the balloon of the urinary catheter.

The solution is not suitable for injection purposes.

Storage: Do not freeze.

Information and specifications on shelf life: **Farco-fill® Protect** should not be longer used after the expiry date.

Keep out of the reach of children.

Supply form: Prefilled syringes with 10 ml **Farco-fill® Protect**.

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Farco-Pharma GmbH