# InstillaGel Lubri

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**Stand der Information:** 2021-01

**Instructions for use.** \_\_\_\_\_ en  
**Please read carefully before use.**

Sterile lubricant gel.

**InstillaGel Lubri** is used in adults, adolescents (12-18 years) and children (2-11 years).

**Composition:** Hydroxyethylcellulose, glycerol, purified water.

**Intended use/indication:** **InstillaGel Lubri** is a sterile gel for use in humans, e.g. during medical procedures to assist the insertion of catheters and endoscopic devices during transurethral, rectal and vaginal examinations, as well as during intermittent catheterisation, in particular for incontinence treatment, and during gastroscopy, intubation and bronchoscopy. It is used as a gel for sonic coupling during sonographic examinations, as a neutral gel base for the preparation of mixtures with other substances for diagnostic and therapeutic purposes, and for use during sexual intercourse.

**Properties:** **InstillaGel Lubri** ensures that catheters and instruments slide easily and adheres well to the mucosa. It leaves the instrument optic clear. **InstillaGel Lubri** is very well tolerated, water-soluble and fat-free.

**Interaction with other medicinal products and other forms of interaction:** There are no known interactions.

**Side effects:** With rectal use, glycerol can have a mild laxative effect.

**Contraindications:** Allergic predisposition to any of the ingredients.

**Warnings and precautions:**

**InstillaGel Lubri** contains glycerol and can therefore have a mild laxative effect if used rectally.

There is no adequate experience or studies of the safety of use of **InstillaGel Lubri** in pregnancy and lactation period. The ingredients suggest, however, that no harm is to be expected.

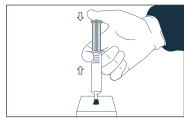
With more prolonged procedures, there may be a change in the viscosity of the gel.

**Use with other products:** The choice of suitable catheters, endoscopes, tubes and specula should be made by a physician.

There is no evidence of undesirable interactions or impaired function between **InstillaGel Lubri** and instruments and medical devices commonly used in the context of the intended use of **InstillaGel Lubri**. It is recommended to clarify the use of **InstillaGel Lubri** especially together with active medical devices with the respective manufacturer.

**Dosage instructions:**

**Preparation:** Remove the syringe from the blister. Before removing the cap, a possible pressure point should be loosened by pressing slightly on the plunger (press the cap against firm resistance). This enables the syringe to be emptied easily and evenly. The cap is then removed as far as possible without touching the syringe tip.

Please note that there is an air bubble in the syringe. This does not affect the functionality of **InstillaGel Lubri**.

**Use:** Unless prescribed/specified otherwise:

- For adults and adolescents: to facilitate the insertion of catheters, endoscopes, tubes and specula into body orifices, 10 ml gel are applied to the mucosa or the instrument to be lubricated. The volume depends on the age and/or the type of procedure and on the anatomy.
- For children (2-11 years) 2-4 ml are recommended. The volume depends on the age and/or the type of procedure and on the anatomy. There are no special dosage recommendations and the dosage is determined by the physician. For **InstillaGel Lubri** dosing remove the air bubble prior to instillation by holding the opened syringe upwards vertically and expressing the air bubble as soon as it is at the upper end of the syringe by pressing the plunger slightly. A small amount of gel will also be expressed. There are marks on the syringe plunger rod. These act as a roughly 1 ml guide when instilling smaller volumes. The graduation has no measurement function.

**Additional information**

**Notes:** **InstillaGel Lubri** is intended for single use and must not be resterilised. Discard leftover gel and do not use the empty syringe for other purposes as germs may be transmitted.

**Storage:** The syringe should be stored in the original packaging (blister in carton) at room temperature dry and protected from light (do not exceed a storage temperature of 25 °C and do not freeze). The product is sterile as long as the blister is sealed and undamaged. Do not use the syringe if the blister is open or damaged.

**Information on shelf life:** **InstillaGel Lubri** must not be used after the stated expiry date. The expiry date refers to the last day of the stated month.

**Disposal:** **InstillaGel Lubri** can be disposed of with domestic waste in accordance with national guidelines. If you are uncertain, ask your doctor or pharmacist how to dispose of the product.

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**FARCO-PHARMA GmbH** — Gereonsmühlengasse 1–11