Two-component hydrogel to remove kidney stone fragments

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The following equipment is needed to use mediNIK®: a sterile syringe containing 5 mL of the "K1" blue component (base substance) and a sterile syringe containing 11 mL of the "K2" yellow component (activator). Each sales unit contains five syringes each of K1 and K2.

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Intended Use: mediNIK® is used to enclose kidney stones and concretions in a hydrogel for removal during ureterorenoscopy (URS). The use of mediNIK® allows fragments to be removed from the kidney that otherwise could not be removed with a grasping instrument due to their small size.

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Users: For use by healthcare professionals only.

Contraindications: Hyperensitivity or allergic reaction to any of the ingredients. The contraindications and restrictions of ureterorenoscopy apply to allergic reaction on one of the ingredients. The contraindications and restrictions of ureterorenoscopy include: coagulation disorders, anticoagulation therapy, sepsis, untreated urinary tract infections, pregnancy, kidney disease, intolerance to endoscopic procedures, marked hyperplasia of the prostate, ureteral stenosis.

Contraindicationer: Hypersensibilitet og reaktion på nogen af ingrediensene. De kontraindikationer og restriktioner af ureterorenoskopie, der er angivet herunder, gælder ikke for alle ingredienser. Kontraindikationer og begrænsninger for ureterorenoskopie (URS) inkluderer følgende: coagulation disorders, anticoagulation therapy, sepsis, untreated urinary tract infections, pregnancy, kidney disease, intolerance to endoscopic procedures, marked hyperplasia of the prostate, ureteral stenosis.

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Combinação com acessórios: There are no known restrictions on the choice of endoscopes, grasping instruments, and ureteral access sheaths (UAS).

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Before use of mediNIK®:

A. Make sure that there are no contrast agents, blood, or other substances in the kidney that could affect visualisation and/or hydrogel formation.

B. Before removing the syringe cap, release the pressure point by pressing the plunger down. This ensures that the syringe can be easily and evenly emptied.

C. For best results, the kidney stones should be gathered at the deepest point of a renal calyx.

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C. Für best results, the kidney stones should be gathered at the deepest point of a renal calyx.

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During use of mediNIK®:

1. Steer the endoscope until the kidney stone fragments are visualised. Flush with sterile saline solution (0.9% NaCl) to ensure good visualisation.

2. Stop flushing. Administer the K1 blue basic substance slowly by pulsing it through the working channel of the endoscope until the fragments to be removed are completely enclosed in the K1 blue substance. If more than one renal calyx is affected, repeat this step for each calyx.

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3. Flush the working channel of the endoscope carefully with 0.9% NaCl.

4. Administrare endoscopul tills nyrestensfragmenten visas. Spül med steril saltvannsoplosning (0,9% NaCl) for å sikre god visualisering.

5. Stop the irrigation! Apply the blue basic substance slowly by pulsing it through the working channel of the endoscope until the fragments to be removed are completely enclosed in the K1 blue substance. If more than one renal calyx is affected, repeat this step for each calyx.

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